



## Oxivir® Tb and Oxivir® Tb Wipes

General Virucide,  
Bactericide,  
Tuberculocide,  
Fungicide,  
Sanitizer



A ready-to-use cleaner disinfectant based on proprietary AHP® technology to deliver fast, effective cleaning performance.

### Features:

- One-step cleaner and disinfectant
- Powered by Accelerated Hydrogen Peroxide technology
- No VOCs, no added fragrance and no NPEs
- Environmentally responsible – active ingredients break down to oxygen and water after use

### Benefits:

- Effective against key pathogens – TB, MRSA, Norovirus and VRE
- 1 minute kill for HBV, HCV, HIV-1 (AIDS virus) , Tuberculocidal kill in 5 minutes and fungi in 10 minutes
- Safe for use around patients, by visitors and on most surfaces
- Convenient – pre-wetted wipes make even the toughest job easier
- 30 second soft-surface sanitizer (RTU only)

### Applications:

- Clean and disinfect high-touch surfaces in just ONE Minute
- Non-irritating to eyes and skin and contains no substances which at the given concentration, are considered to be hazardous to health per OSHA standards



# Oxivir® Tb and Oxivir® Tb Wipes

General Virucide, Bactericide, Tuberculocide, Fungicide, Sanitizer



## Use Directions: As a One-Step Cleaner Disinfectant

1. Pre clean heavily soiled areas
2. Apply product by coarse trigger spray or disposable wipe to hard, non-porous inanimate surfaces
3. All surfaces must remain visibly wet for 1 minute
4. Use 5 minute contact time for Tuberculosis and a 10 minute contact time for fungi
5. Wipe surfaces dry (or rinse) (or allow to air dry)

**Note:** Rinsing is not necessary unless floors are to be coated with finish or restorer. All food contact surfaces must be rinsed with potable water. Do not use on glassware, utensils, or dishes. Not recommended for use on brass, copper, aluminum, bronze, granite, zinc or marble.

## Use Directions: As a Soft Surface Sanitizer (RTU only)

1. Remove gross filth prior to sanitization
2. Hold bottle upright 6" – 8" from surface
3. Spray until fabric is wet. **DO NOT SATURATE**
4. Allow surface to remain wet for 30 seconds
5. Gently blot with a clean, damp, color safe cloth and/or let air dry
6. For difficult odors or heavy fabrics, repeat application
7. Repeat as needed

**Note:** Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility.

## Product Specifications

Description	
Color/Form	Colorless, liquid; Colorless, wipe
EPA Reg. No.	70627-56 (RTU) 70627-60 (Wipes)
pH	2.5–3.5
Scent	Characteristic
Solubility	Complete
Shelf Life	2 years (RTU), 3 years (Wipes)
Specific Gravity	1.01

## Available Items

Product Code	Description/Package Size	Dilution
4277285	12 x 32 fl. oz. / 946 mL Flip Top Bottles Plus Two Trigger Sprayers	Ready-to-Use
4599516	12 x 160 count 6 x 7 in. Wipes / 15.24 x 17.78 cm Canisters	Ready-to-Use
5388471	12 x 60 count 7 x 8 in. Wipes / 17.8 x 20.3 cm Canisters	Ready-to-Use
5627427	4 x 160 count 11 x 12 in. Wipes / 27.9 x 30.5 cm Buckets	Ready-to-Use
100823906	4 x 160 count 11 x 12 in. Wipes / 27.9 x 30.5 cm Refills	Ready-to-Use
5499664	6 x 1 Unit (Wall Bracket for use with 60 count Canisters)	
9P700798	1 Double Unit Non-Locking Stand	
9P700797	1 Double Unit Locking Stand	
9P700814	1 Plastic Locking Bracket (for use with 160 count Buckets)	
9P700799	6 x 1 Unit Wire Bracket Locking on Plastic Wall Mount (for use with 160 count Buckets)	
9P700800	6 x 1 Unit Wire Bracket Non-Locking on Plastic Wall Mount (for use with 160 count Buckets)	

## Safety Reminder

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use, and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDSs are available on-line at [www.sealedair.com](http://www.sealedair.com) or by calling 888 352 2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up to date shipping information.

AHP and Design® are registered trademarks of Virox Technologies, Inc. used under license.

[www.sealedair.com](http://www.sealedair.com)

© 2015 Sealed Air Corporation. All Rights Reserved. 34206-PSS en (15/196)